




**Customizable Visco Cushion**

Our Visco Cushion provides optimal pressure reduction for bony prominences on the ischiums/sitter bones and true pressure relief for the coccyx area is achieved when the *\*Customizable* option is applied.

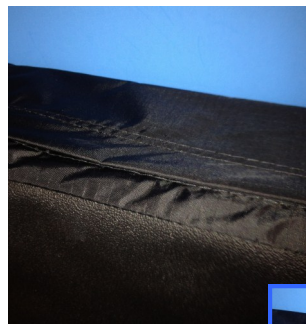

The machine washable/dryable cover is constructed of Ripstop material with a non skid bottom that includes flame retardant, anti- bacterial, and fluid resistant properties. The non skid bottom and attached ties help secure cushion in place whether in a wheelchair, office chair, recliner or automobile seat.

Zipper is concealed to protect cushion against incontinence issues



| Item No. | Dimension         | Case Qty. |
|----------|-------------------|-----------|
| 118-5002 | 18"W x 16"D x 2"H | 12        |
| 118-5003 | 18"W x 16"D x 3"H | 12        |
| 118-5004 | 18"W x 16"D x 4"H | 12        |

Also shown in wheelchair: Kyphosis Cushion

| Item No. | Dimension            | Case Qty. |
|----------|----------------------|-----------|
| 118-9100 | 19.75"H x 16"W x 3"D | Each      |

**\*To customize your Visco Cushion:**

- Remove the cover
- While sitting on the cushion, locate the area that requires pressure relief (using a marker of some sort to identify the foam cubes)
- With area marked, use scissors or other sharp object to cut the base of the foam cubes and remove
- Reinsert cushion into cover (grid pattern should be facing up), paying close attention to the direction of the customized area(s)

